KING PLASTIC RIBBED GRAB BARS

30 CM/12IN, 40 CM/16IN, 45 CM/18IN, 60 CM/24IN

EN USER MANUAL
DE BENUTZERHANDBUCH
FR MANUEL DE L'UTILISATEUR
ES MANUAL DE USUARIO
IT MANUALE D'USO
PT MANUAL DO USUÁRIO
SL NAVODILA ZA UPORABO
HR KORISNIČKI PRIRUČNIK
HU HASZNÁLATI UTASÍTÁS
RO MANUAL DE UTILIZARE

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30 CM/12IN, 40 CM/16IN, 45 CM/18IN, 60 CM/24IN

INTENDED USAGE:

The products of the *Plastic Ribbed Grab Bars KING* allow to increase independence while taking a shower or while standing up and sitting down on the toilet bowl or in the bathtub. They work as a robust and safe handle, on which the user can hold to relieve part of the body weight from the legs, reducing muscle fatigue due to standing position as well as slip and injury risks. Depending on the user's requirements, they can be installed in different places in the bathroom.

INDICATIONS AND CONTRAINDICATIONS:

The products of the Plastic Ribbed Grab Bars KING line are aimed towards elderly people, wheelchair users, and all those who need further support while using the bathroom. The user is required to have sufficient ability to grip firmly the grab bar as well as sufficient autonomy.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the grab bar is firmly fixed to the wall.
- During the assembly, strictly adhere to what is shown in the User Manual.
- Do not modify the grab bar in any way.
- Do not use the grab bar if damaged or altered in any way.
- Do not touch the grab bar with greasy hands. A slippery surface may reduce the grip and could cause a fall.
- Do not use the grab bar for a different purpose from the intended one: it must not be used to replace a stair.
- The grab bar must not be stored outdoors.

MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at info@kingbathrooms.com and the responsible authority in your country.

ITEM CODE	Description	PAGE
KING-GB30R-KN	Plastic ribbed grab bar 30 cm/12in King	44-45
KING-GB40R-KN	Plastic ribbed grab bar 40 cm/16in King	44-45
KING-GB45R-KN	Plastic ribbed grab bar 45 cm/18in King	44-45
KING-GB60R-KN	Plastic ribbed grab bar 60 cm/24in King	44-45

ICON	DESCRIPTION
	Made in Italy
	Easy to assembly
	Max User Weight
	2 Years Warranty

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LM Global Design Ltd.