
430910 Small 105cm
430911 Medium 120cm
430912 Large 135cm
430913 Xlarge 150cm

430910	S	105cm	RED
430911	M	120cm	YELLOW
430912	L	135cm	GREEN
430913	XL	150cm	BLUE

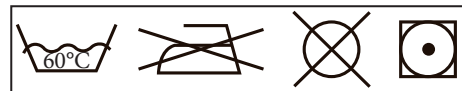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Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Do not store outside and do not let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ No modification or repair must be made on this device.
- ♥ Follow the advice of the professional who prescribed or supplied the product.
- ♥ This device was prescribed by a professional and fits you, it should not be used by another person.
- ♥ Consult your physician in case of discomfort.

2. Indication

TRANSAC is meant to help caregivers during patient transfers as it reduces the strain on the caregiver's back.

3. Description and fitting instructions

The belt goes around the patient's waist and can be used to secure or help them with walking, help the caregiver lift them up, or even transfer the patient from one location to another. This provides the caregiver something to hold on to when assisting a patient out of a chair, making transfer smoother and reducing strain on the caregivers' back.

The belt can be used reversed, i.e. wrapped around the caregivers' waist to enable the patient to grab it while standing up.

Wrap the belt around the waist, adjust it and close it using the safety lock (a). The patient/caregiver can then grab the handles (b) or (c) or both, see photo [1]. To unlock the belt, press on the red dot.

Use the colour of the strap (d) to choose the right size.

4. Washing instructions

Wash with soapy water at maximum 40°C. Rinse thoroughly with lukewarm water. Squeeze out excess water. Dry flat and away from any source of heat. Do not use detergent or bleach. Do not iron.

5. Composition

PA, PP, PES, PE,PU, PVC (see product label).

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

This product has a 2-year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible and CE certified.
Duration of use: 2 years.

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