
Marks	Designations	A	B	C	D	E	F	G	H		
30040	New Open + full seatplate	85.5 cm	56 cm	63 cm	71 cm	50 cm	40 cm	39 cm	49 cm	6.6 kg	130 kg
30042	New Open + standard seatplate										
3003**/3005**	Open + standard seatplate/+ full seatplate	86 cm	51.5 cm	60.5 cm	72 cm	52 cm	38 cm	39 cm	58 cm	6.6 kg	
30100	New Open adjustable + full seatplate	82.5/97 cm	55.5 cm	62.5 cm	67.5/83 cm	46/62 cm	40 cm	42 cm	49/52 cm	6.8 kg	
30102	New Open adjustable + standard seatplate										
3015**	Open Adjustable + standard seatplate	86/97 cm	51.5 cm	60.5 cm	72/83 cm	52/63 cm	38 cm	39 cm	58 cm	6.8 kg	
3014**	Open Adjustable + full seatplate										
3200**	New club + standard seatplate	84 cm	53 cm	61 cm	73 cm	52 cm	35 cm	40 cm	49 cm	7 kg	
3201**	New club + full seatplate										
3046**	Best Up	94.5 cm	44 cm	47 cm	54 cm	48.5 cm	50 cm	39.5 cm	45 cm	9.4kg	160 kg
3646**	Best Up XXL	88 cm	75 cm	87 cm	75 cm	56 cm	59 cm	44 cm	59 cm	9.5kg	
	Dimension of the hole: Width = 21 cm Depth = 25.5 cm										

	FR	EN	DE	NL	IT	ES
	Signification des symboles	Explanation of symbols	Zeichenerklärung	Verklaring van de symbolen	Significato dei simboli	Explicación de los símbolos
1	Dispositif médical	Medical Device	Medizinprodukt	Medisch hulpmiddel	Dispositivo Medico	Dispositivo médico
2	Lire le mode d'emploi avant l'utilisation.	Read the instruction manual before use	Vor Gebrauch Gebrauchsanweisung lesen!	Voor gebruik de handleiding lezen.	Prima dell'uso leggere le relative istruzioni!	Lea el manual de instrucciones antes de utilizarlo
3	Respecter les instructions de sécurité!	Observe the safety instructions!	Sicherheitshinweise beachten!	Veiligheidsinstructies respecteren	Osservare le istruzioni per la sicurezza	Siga las instrucciones de seguridad
4	Conformité CE	CE conformity	CE-Konformität	CE conformiteit	CE conformità	Conformidad con la normativa CE
	SI	HR	HU	PT	PL	TR
	Pomen simbolov	Objašnjenje simbola	Szimbólumok jelentése	Significado dos símbolos	Objaśnienia symboli	Sembollerin açıklamaları
1	Medicinski pripomoček	Medicinski uređaj	Orvosi eszköz	Dispositivo médico	Wyrób medyczny	Tıbbi cihaz
2	Pred uporabo preberite navodila	Pročitati korisničke upute prije uporabe	Használat előtt olvassa el a használati útmutatót	Antes de utilizar, leia, por favor, o manual.	Przed użyciem zapoznaj się z instrukcją obsługi	Kullanmadan önce kullanma kılavuzunu okuyunuz
3	Upošteвайте varnostna navodila!	Pažljivo pogledajte sigurnosne upute	Tartsa be a biztonsági utasításokat!	Siga as instruções de segurança	Stosuj się do instrukcji bezpieczeństwa.	Güvenlik uyarılarını dikkate alınız
4	CE Certifikat	CE-Deklaracija	CE megfeleléség	Certificado CE	Deklaracja CE	CE uygunluğu

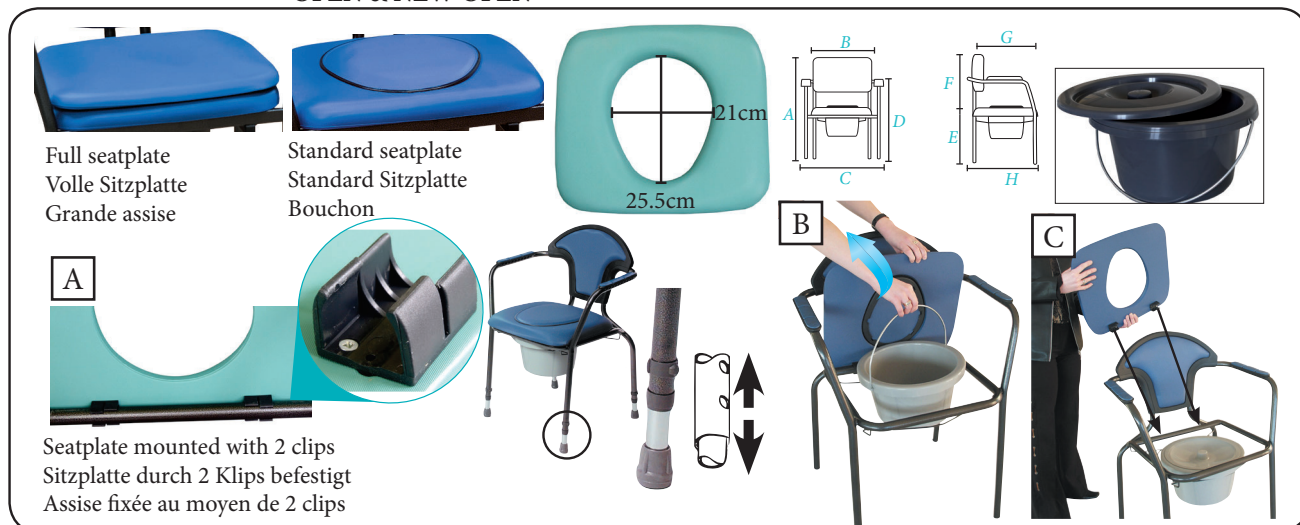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EN- Please read these safety instructions carefully before use, and keep them for future question.

1.Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Before seating make sure that you have correctly placed the seatplate on the chair.
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips)..
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a commode chair and in proper way : don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
- ♥ DO NOT hang anything on the Chair.
- ♥ ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
- ♥ Do not store outdoor, the chair must be used at home.
- ♥ Do not install or remove the pan while the patient is seating on the chair! High risk of sex injuries.
- ♥ Be carrefull when removing the armrests to avoid trapping your fingers.
- ♥ Check the chair before using, in case of doubt ask your physician or provider.
- ♥ Use only the chair on a flat and solid ground.
- ♥ It is forbidden to seat on the armrests.
- ♥ Do not lean out of the chair to avoid losing balance.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member

State in which the user and/or the patient is established.

2. Intended use, Indication and Contraindication

Commode chairs can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by bringing the toilet into the bedroom. Standard, foldable or on castors, they shall be use only indoor.

Indication

Used in the home when the patient is too debilitated to reach the bathroom.

Contraindication

Don't use the commode chair in case of:

- Cognitive disorder - Loss of balance - Sitting impairment

3. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Installing/removing the bucket

Open & Best up:

1- from the left or right side under the seat, install the bucket onto the metal supports (only possibility for XXL models).

2- lift up the seatplate and remove the bucket from the top, see photo [B].

New Open, New Club:

Lift up the seatplate and remove the bucket from the top, see photo [B].

Close the bucket with the lid before transporting it.

Mounting the seatplate

The seatplate can be easily removed for cleaning or for replacement as it is only connected to the frame with 2 clips see photo [A].

To remove the seatplate, take out the clips from the frame by lifting them up, see photo [B].

To mount the seatplate, clip the two clips on the back pipe of the metal frame, see photo [C].

By adding the cover on the hole, the commode chair can be used as a regular chair.

Rental: This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

1-Visual inspection: ensure that the product has not been damaged and check for any defect.

2-Functional inspection: ensure that the product functions properly and repair it if needed.

3-Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories). Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product.

Use only appropriate tools to perform the maintenance, cleaning and disinfection.

The user's manual is an integral part of the product and must be handed over to every new user.

4. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

5. Materials

The product is made up of steel tubing covered with epoxy painting. Upolsteries are in PVC. The chair is delivered with a pan. Some models are equipped with swinging armrests.

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

See tabel page 1

This product has a 2 years warranty, form the date of purchase, against any manufacturing defect. This product is biocompatible. Duration of use: 2 years.

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